



As we age we begin to notice our skin showing signs of time, sun damage and lifestyle choices. These facial imperfections can detract from our appearance and well-

being. Photorejuvenation offers dramatic improvement to multiple conditions creating younger, smoother and healthier looking skin with little or no downtime.

## Lumenis One Photorejuvenation One:

The Next Generation in State of the Artistry  
Results for Treating:

- Spider Veins
- Age Spots
- Sun Spots
- Sun Damage
- Mottled Pigmentation
- Ruddy Complexion
- Freckles
- Rosacea



For more information on IPL solutions for your skin concerns, consult your physician and visit

[www.skinandhealth.com](http://www.skinandhealth.com)

### Clinical Photography courtesy of:

Patrick Bitter, Jr. M.D. - *Campbell, California, USA*

Jerry Darm, M.D. - *Lake Oswego, Oregon, USA*

Robert Weiss, M.D. - *Hunt Valley, Maryland, USA*



## Photorejuvenation One

The Next Generation of IPL<sup>®</sup> Skin Treatments  
Using Photorejuvenation

**Only Available with the Lumenis One™  
Multi-Technology Platform**



## **What Makes Photorejuvenation One with Lumenis One Special?**

Lumenis One® is a true fourth generation Intense Pulsed Light system with patented state of the art technology which optimizes power, pulsing and light delivery to provide maximum results, comfort and safety. Lumenis invented IPL and continues to perfect it. With more clinical papers and more physicians using Lumenis IPL systems, Lumenis IPL is the Gold Standard, and Lumenis One is the flagship system.


Advanced technology can be expensive. While other practices may promote similar treatments with cheaper, lower-powered technology, there is a visible difference in results. Our practice believes in providing only the very best treatment for our

patients and has made the investment in the Lumenis One.



## **How Does Lumenis One Work?**

Lumenis One delivers a special range of wavelengths of light in customizable pulses through a cold chilled sapphire tip gently onto your skin. The light is absorbed either superficially by broken capillaries and dark pigment or deeper by larger vessels and collagen. Dilated capillaries coagulate, pigmented lesions darken and then slough off and new collagen fibers are laid down, giving the skin a renewed and more youthful appearance.

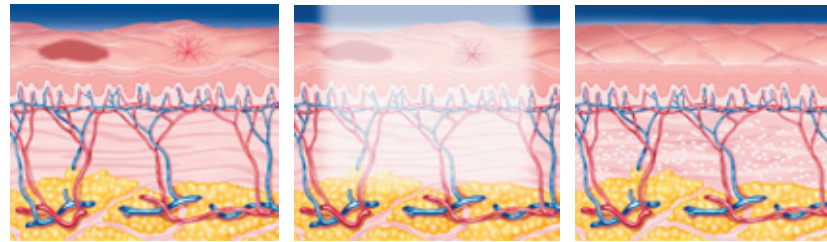

Before Treatment

During Treatment

After Treatment

## **What Areas Can Be Treated?**

Photorejuvenation-One with the Lumenis One system can be used to treat any area of the body. The most common areas are the face, neck, chest and hands.

## **How is Photorejuvenation Performed?**

A cold gel is usually applied to the area to be treated, and you may be given dark glasses or goggles to protect your eyes from the bright light. The smooth, cold surface of the Lumenis One IPL handpiece is gently touched to your skin and pulses of light are applied. You may feel a slight stinging like the snapping of a rubber band. An anesthetic cream may be used, but is seldom required. Each treatment typically takes about 20 minutes. In general, excellent long term results can usually be achieved with a series of four to six treatments.

## **Why Are Multiple Treatments Necessary?**

Depending on the number of conditions you are treating and the severity of each, a series of four to six treatment sessions may be recommended. You can return to work the same day and resume all your regular activities. By dividing the full program into several treatments, the procedure provides gradual improvement with very low risk, preserving the convenience of a no downtime procedure.

Before & After  
(three treatments)  
Sun Damage



Before & After  
(two treatments)  
Ruddy Complexion

